

WOUND CARE DryMax[®]

SUPER

DryMax Super is a sterile, conformable, superabsorbent dressing for use on exuding wounds. DryMax Super absorbs and retains exudate in the dressing. The exudate is absorbed vertically by the dressing and retained in the core forming a gel, thus minimizing the risk of maceration and maintaining a moist wound environment.


DryMax Super

The seamless cover simplifies positioning over the wound. The design allows the dressing to be applied with either side facing the wound.


This feature also makes it possible to stack dressings on top of each other if extra absorption capacity is required.

DryMax Super remains dry and comfortable for the patient. Simultaneously it contributes to a moist wound environment. The hydrophilic contact layer transports fluid effectively away from the wound and distributes it evenly into the ABSORBEST core.


The ultra-thin and highly conformable features allows the dressing to be used on all parts of the body. DryMax Super is designed to work very well together with compression therapy.

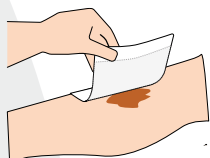

The dressing must be placed close to the wound, or placed in close combination with a suitable dressing.

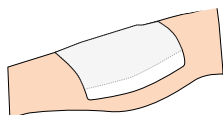

Secure the dressing with suitable bandage, film or tape.

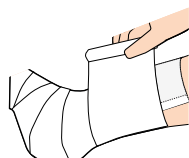

DryMax Super can be used in conjunction with compression therapy.

Size	Dressings per box	Absorption capacity*	Product REF	PIP Code	NHS Cat Number
11x10 cm	10	130 ml	F60073/10	421-3369	EJA293
11x20 cm	10	330 ml	F60075/10	421-3377	EJA288
20x20 cm	10	600 ml	F60076/10	421-3385	EJA285
20x30 cm	10	960 ml	F60077/10	421-3393	EJA294
37x56 cm	10	3000 ml	F60079/10	421-3401	EJA283

*) According to international standard EN 13726



- User friendly
- Increased patient comfort
- Improved wound healing
- Cost effective



Manufactured by Absorbent AB Sweden
www.absorbent.se | info@absorbent.se