

Application example for DryMax Super 37x56 cm

DryMax Super is a sterile, conformable, superabsorbent dressing for use on exuding wounds. The design allows the dressing to be applied with either side facing the wound. The dressing remains dry and comfortable for the patient. Simultaneously it contributes to a moist wound environment.

The ultra-thin and highly conformable features allows the dressing to be used on all parts of the body. DryMax Super absorbs and retains exudate in the dressing. The exudate is absorbed vertically by the dressing and retained in the core forming a gel, thus minimizing the risk of maceration and maintaining a moist wound environment. DryMax Super is designed to work well together with compression therapy.

Item	NHS Cat Number	PIP Code	Pcs/box	Product size (cm)	Absorption capacity*
DryMax Super	EJA283	4213401	10	37x56 cm	3000 ml

*) According to international standard EN 13726

Leg



1. Place the dressing under the leg.



2. Fold the dressing from both sides at the malleolus to cover the foot.



3. Continue to fold the dressing around the entire leg.



4. Make sure the wound area is covered. Use tape to hold the folded boot shape in place.



5. Pass a tube gauze as a sock over the dressing.



6. The dressing is in place.

Arm



1. Place the dressing around the arm so that the wound area is covered.



2. Thread on a tube gauze over the arm.




3. The dressing is in place.

Groin, thigh and hip



1. Place the dressing over the thigh or hip so that the wound area is covered. If necessary, the dressing can be folded and adjusted to the contours of the body.

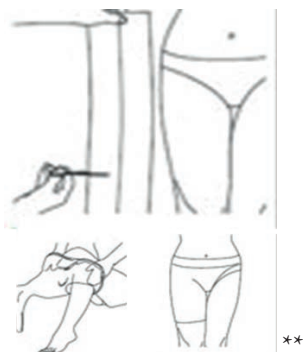

2. Make one cut in the middle of the tube gauze. Thread on the tube gauze like pants.




3. The dressing is in place.

Chest



1. Place the dressing over the chest so that the wound area is covered. If necessary, the dressing can be folded and adjusted to the contours of the body.

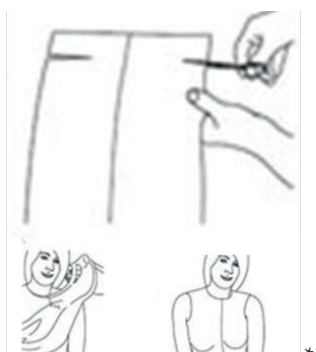

2. Make one cut on each side of a tube gauze to make holes for the arms. Thread on the tube gauze as a sweater over the dressing.




3. The dressing is in place.

Back



1. Place the dressing over the back so that the wound area is covered. If necessary, the dressing can be folded and adjusted to the contours of the body.

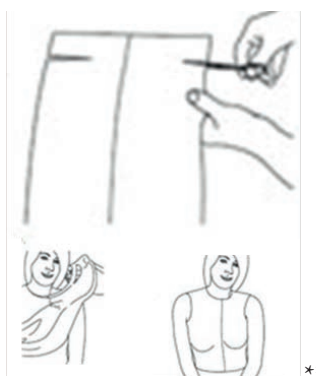

2. Make one cut on each side of a tube gauze to make holes for the arms. Thread on the tube gauze as a sweater over the dressing.




3. The dressing is in place.

Abdomen



1. Place the dressing over the abdomen so that the wound area is covered. If necessary, the dressing can be folded and adjusted to the contours of the body.

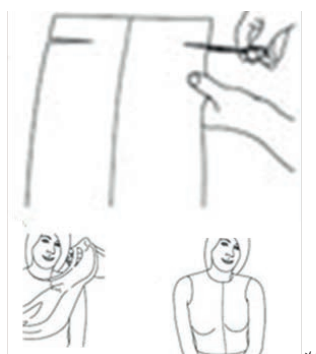

2. Make one cut on each side of a tube gauze to make holes for the arms. Thread on the tube gauze as a sweater over the dressing.



3. The dressing is in place.

***) Example from Mölnlycke Tubifast