

DryMax Super

DryMax Super is a sterile, conformable, superabsorbent dressing for exuding wounds. Its properties provide an easy-to-use and reliable solution for both patient and clinician. DryMax Super absorbs and retains exudate in the dressing. The exudate is absorbed vertically by the dressing and retained in the core, forming a gel, thus minimizing the risk of maceration and maintaining a moisture balanced wound environment.

The seamless cover simplifies positioning over the wound. The design allows the dressing to be applied with either side facing the wound. This feature also makes it possible to stack dressings on each other if extra absorption capacity is required.


The dressing remains dry and comfortable for the patient. Simultaneously, it contributes to a moist wound environment. The hydrophilic contact layer transports fluid effectively away from the wound and distributes it evenly into the dressing's core constructed by our DryMax Technology.

The ultra-thin and highly conformable features allow the dressing to be used on difficult external body parts. DryMax Super is designed to work well together with compression therapy.


Application Guide



The dressing must be placed close to the wound, or placed in close combination with a suitable dressing.

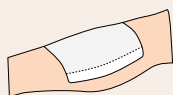

Secure the dressing with suitable bandage, film or tape.

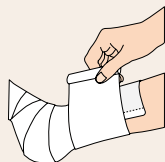

DryMax Super can be used in conjunction with compression therapy.

Product size cm	REF	Pcs/box	Absorption-capacity*	PIP Code	NHS Cat Number
5x5	F60125/10	10	20 ml	433-8075	ELZ85079
11x10	F60073/10	10	130 ml	421-3369	EJA293
11x20	F60075/10	10	330 ml	421-3377	EJA288
20x20	F60076/10	10	600 ml	421-3385	EJA285
20x30	F60077/10	10	960 ml	421-3393	EJA294
37x56	F60079/10	10	3000 ml	421-3401	EJA283



For further information, or to request a product sample scan here

*According to international standard EN 13726

**Compared to conventional wound dressings



- Cost effective
- User friendly
- Increased patient comfort
- Improved wound healing**



DryMax® technology

Proud to be the UK distributor of DryMax



01942 816184
info@cdmedical.co.uk
www.cdmedical.co.uk



Manufactured by Absorbest AB
Sweden
www.absorbest.co.uk